

AUTO CHARGE DC CABLE



MODEL #: BPS DC.CABLE



877-445-5228

www.batterypowersolutions.net



BPS Auto Charge DC Cable

The auto charge DC cable allows you to safely and efficiently charge the Freedom™ or C-100 CPAP Battery via the cigarette lighter/power socket of a vehicle. The vehicle must be running during the charging process to ensure the battery receives proper voltage to successfully charge.

PACKAGE CONTENTS

Auto Charge DC Cable | Connector Tip

SPECIFICATIONS

Input/Output: 12V DC | Length: 51.5" (end to end) | Weight: 3 oz


SINGLE BATTERY CONNECTION

- Insert the connector tip into the auto charge DC cable.
- Plug the auto charge DC cable with connector tip attached into the DC IN jack of the battery.
- Turn the vehicle on and plug the auto charge DC cable into the cigarette lighter/power socket of the vehicle to begin the charging process.
- When charging is complete disconnect the auto charge DC cable from the vehicle and the battery.


DUAL BATTERY CONNECTION

- Insert the connector tip into the auto charge DC cable.
- Plug the auto charge DC cable with connector tip attached into DC power pigtail cord included with the CPAP battery kit.
- Plug the DC power pigtail cord into the DC IN jacks of both batteries.
- Turn the vehicle on and plug the auto charge DC cable into the cigarette lighter/power socket of the vehicle to begin the charging process.
- When charging is complete disconnect the auto charge DC cable from the vehicle and the battery.

SINGLE BATTERY CONNECTION

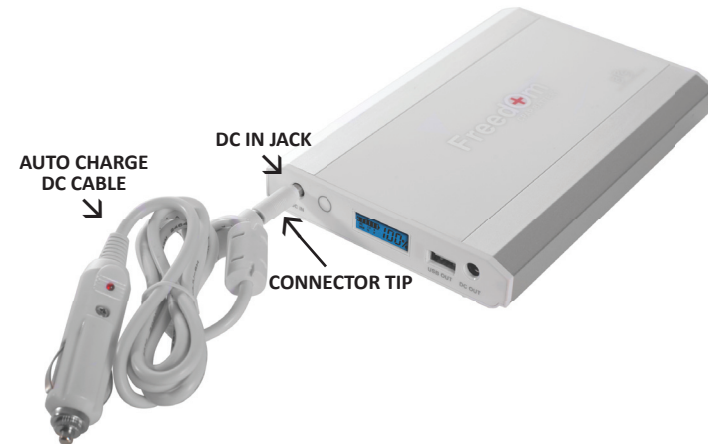

DUAL BATTERY CONNECTION

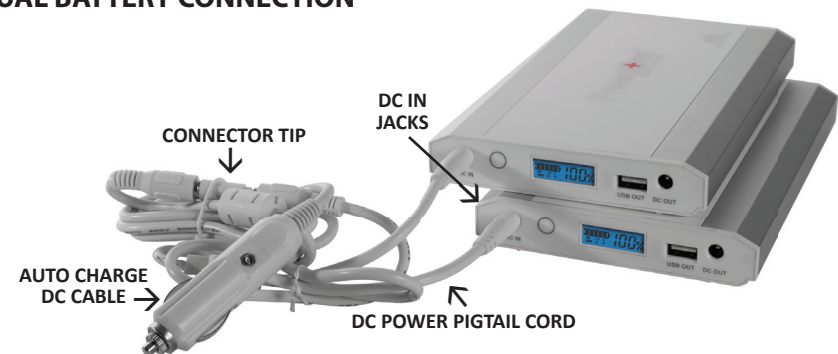

NOTES

- The typical charge time when the battery is fully discharged is 4 hours (per battery). The charge time may vary depending upon the vehicle's battery type.
- When charging any electronic device or external battery in a vehicle it is recommended that the vehicle be running during the charging process to help prevent draining the vehicle's battery.