

BPS 12V Power Adapter for Respiration DreamStation Series Devices

INTRODUCTION

While Respiration DreamStation Series devices operate from 12V direct current, the DC port on these devices require a proprietary signal in order to recognize a valid power source; therefore, the use of a 12V power adapter is required in order for the CPAP Battery to power these devices.

Battery Power Solutions' 12V Power Adapter was specifically designed for use with Respiration DreamStation Series devices and the C-100 CPAP Battery.

SPECIFICATIONS

Input/Output: 12V DC

Length: 66" (connector to connector)

Weight: 4 oz

CONNECTION INSTRUCTIONS

- Insert the 12V power adapter into the DC out jack of the Battery.
 - Dual Battery Configuration:
 - Insert the DC power pigtail cord that is included with the CPAP Battery Kit into the DC out jacks of both Batteries.
 - Connect the 12V power adapter to the female end of the DC power pigtail cord.
- Insert the 12V power adapter into the DC input port of the Respiration DreamStation Series device.
- Turn on the PAP device and it should begin functioning.

SINGLE BATTERY CONFIGURATION



DUAL BATTERY CONFIGURATION

