INSTRUCTIONS FOR USE

185 044 542 Rev D

CONTENTS

English

Français (French)

Español (Spanish)

FISHER & PAYKEL HEALTHCARE LTD	
INTERNATIONAL	15 Maurice Paykel Place, East Tamaki, Auckland 2013 PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.com Website: www.fphcare.com
AUSTRALIA	Tel: +61 3 9879 5022 Fax: +61 3 9879 5232
AUSTRIA	Tel: 0800 29 31 23 Fax: 0800 29 31 22
BENELUX	Tel: +31 40 216 3555 Fax: +31 40 216 3554
CHINA	Tel: +86 (20) 3205 3486 Fax: +86 (20) 3205 2132
FRANCE	Tel: +33 1 6446 5201 Fax: +33 1 6446 5221
GERMANY	Tel: +49 7182 93777 0 Fax: +49 7182 93777 99
INDIA	Tel: +91 80 4123 6041 Fax: +91 80 4123 6044
IRISH REPUBLIC	Tel: 1800 409 011
ITALY	Tel: +39 06 7839 2939 Fax: +39 06 7814 7709
JAPAN	Tel: +81 3 3661 7205 Fax: +81 3 3661 7206
MIDDLE EAST	Tel: +9 71 4 368 0849 Fax: +9 71 4 368 8110
SPAIN	Tel: +34 902 013 346 Fax: +34 902 013 379
SWEDEN	Tel: +46 8 564 76 680 Fax: +46 8 36 63 10
SWITZERLAND	Tel: 0800 83 47 63 Fax: 0800 83 47 54
UK	(EU Authorised Representative) Fisher & Paykel Healthcare Ltd Unit 16, Cordwell's Park, Chisemont Road, Maidenhead, Berkshire SL6 7BL, United Kingdom Tel: +44 1628 626 136 Fax: +44 1628 626 146
Northern Ireland	Tel: +0800 132 189
USA	Tel: +1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

CE 0123

© Fisher & Paykel Healthcare Limited

FLEXIFIT™ 432 FULL FACE MASK INSTRUCTIONS

The FlexiFit™ 432 Full Face Mask is intended for multiple patient or single patient adult use by individuals who have been diagnosed by a physician as requiring CPAP or Bi-Level ventilator treatment in the home, hospital or other clinical setting.

FITTING YOUR MASK

Refer to the mask assembly diagram.

- Remove plastic cover from Silicone Seal.
- Unclip the Headgear (K) from the Gilder™ Strap (E) by pulling the Release Cord (I) down. Holding your Mask Base (A) in one hand, spread the Headgear with your free hand, just enough to fit comfortably over your head.
- Lightly hold the mask against your face, with the bottom of the Silicone Seal under your chin, and spread the Headgear over your head with your free hand.
- Reattach the Headgear Clip (H) to the Gilder™ Strap.
- Gently adjust the top horizontal straps, then the lower horizontal straps, then the top crown straps. Ensure the top of the Silicone Seal remains stable on the bridge of the nose, and the bottom remains fixed under the chin. Ensure that the Gilder™ Strap is centered.
- Connect the CPAP/Bi-Level machine hose running at the prescribed pressure to the mask.

FITTING TIPS

- If necessary, pull the mask slightly away from your face while the straps are still attached. This allows the Silicone Seal to inflate. Put the mask back on your face.
- If leaks occur around the upper half of the mask, gently tighten the top horizontal straps. It is not necessary for the Forehead Cushion to touch your forehead.
- If leaks occur around the lower half of the mask, gently tighten the lower horizontal straps. (Avoid over-tightening the straps.)
- When you need to remove your mask unclip the Headgear from the Gilder™ Strap by pulling the Release Cord forward or down.

CLEANING YOUR MASK AT HOME

To ensure continuing safety and comfort for your FlexiFit™ 432 Full Face Mask:

BEFORE FIRST USE:

- Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight. It is not necessary to remove the Clip (H) from the Headgear when washing.

BEFORE EACH USE:

- Inspect the mask for deterioration. Do not use if damaged.
- Inspect the Non-Rebreathing Valve (J). Ensure the silicone flaps are down when the CPAP/Bi-Level machine is turned off.

AFTER EACH USE:

- Disassemble the Silicone Seal (C), Foam Cushion (D), Elbow (B), Non-Rebreathing Valve (J), and the Headgear (K) from the Mask Base (A).
- Wash the Silicone Seal, Elbow, Valve and Mask Base in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water.
- Wipe the Foam Cushion (D) with a damp cloth (do not immerse the Foam Cushion in water).
- Before reassembling, leave all parts to dry away from direct sunlight.

EVERY 7 DAYS:

- Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to air dry out of direct sunlight. It is not necessary to remove the Clip (H) from the Headgear when washing.

CAUTION

- Do not soak for longer than 10 minutes.
- Do not wash the mask in the dishwasher.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ 432 Full Face Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your FlexiFit™ 432 Full Face Mask is fully assembled and ready to use. However, when you need to clean your mask, follow these instructions for reassembly:

- Place Silicone Seal (C) face down on a flat surface. With the thick side of the Foam Cushion (D) facing down, place onto Silicone Seal.
- Place Silicone Seal (A) and Foam Cushion (D) into the Mask Base (A).
- Push the Non-Rebreathing Valve (J) onto the Elbow (B), and attach to Mask Base.
- Lay the Headgear (K) flat with the label side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Mask Base and the Gilder™ Strap (E). This can be done without undoing the Velcro™ Tabs by sliding the Headgear into the slots.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC4S2).

- Connect the large end of the Connector (T) to the Non-Rebreathing Valve (J) on the mask and the small end of the Connector to the Breathing Tube (R).
- Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the Port(s) (Q).

(The two Ports are identical; either Port may be used for oxygen or pressure measurement. Only remove the Port Cap from Ports that are being used.)

- Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

- Warning: If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.
- Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.
- Warning: Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- Warning: Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen tubing. Use silicone tubing instead.
- Explanation of Warning: Contact with PVC can cause premature cracking of ports.

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. The exhalation port(s) associated with the mask should never be blocked.

Explanation of Warning: CPAP and Bi-Level machines are intended to be used with special masks which have exhalation ports to allow continuous flow of air out of the mask. When the CPAP or Bi-Level machine is turned on and functioning properly, new air from the CPAP or Bi-Level machine flushes the exhaled air out through the exhalation port(s). However, when the CPAP or Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.

- At low CPAP/Bi-Level pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Valve activation/deactivation pressure is 2 cm H₂O.
- Do not use the mask without the Non-Rebreathing Valve in place. This valve ensures CO₂ re-breathing is kept to a minimum in the event of power failure.
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If using the FlexiFit™ 432 Full Face Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the FlexiFit™ 432 Full Face Mask.
- Replace the Non-Rebreathing Valve every 12 months.
- Do not use the FlexiFit™ 432 Full Face Mask Elbow on other Fisher & Paykel Healthcare masks.
- Use the FlexiFit™ 432 Full Face Mask only with the Headgear provided.

CAUTION

- Use the mask for its intended use only as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of a physician.

CONTRAINDICATIONS

- This mask should not be used by anyone incapable of removing the mask unaided, without continuous supervision.

The FlexiFit™ 432 Full Face Mask and accessories are Latex free.
Many people find CPAP therapy causes dryness of the nose and throat.
Heated humidification can substantially reduce these symptoms.
Contact your supplier and ask about ThermoSmart™.

US Patent Numbers 6615834 / 6789541 / 6701926 / 6951218
Other Patents Pending

TECHNICAL SPECIFICATIONS

EXHAUST FLOW												
The FlexiFit™ 432 Full Face Mask has exhalation ports to expel the air that you exhale from the mask.												
It is important that these exhalation ports are not blocked by any object. This controlled leak ensures all exhaled CO ₂ is expelled from the mask.												
Pressure (cmH ₂ O)	3	5	7	9	11	13	15	17	19	21	23	25
EXHAUST FLOW (L/MIN)	21	24	28	31	36	39	41	44	47	50	52	54
RESISTANCE TO FLOW												
Pressure drop through mask @ 50 l/min 0.3 cmH ₂ O												
MASK DEAD SPACE												
115cc												
OPERATING CONDITIONS												
<ul style="list-style-type: none"> Operating pressure is 3-25 cm H₂O Operating temperature range is 40-120°F (5-50°C) 												

FLEXIFIT™ 432 FULL FACE MASK MULTI-PATIENT CLEANING INSTRUCTIONS

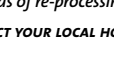
Your FlexiFit™ 432 Full Face Mask is provided patient ready to reuse between patients please follow the guidelines below.

CLEANING / DESOILING

- Before disinfecting, disassemble all components from the Mask Base. Clean all surfaces of mask and components thoroughly with a mild detergent solution and a soft bristled brush. Ensure all visible soil is removed. Rinse thoroughly under running potable water to remove detergent solution.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION

HIGH LEVEL DISINFECTION PARAMETERS

 MASK FRAME	THERMAL DISINFECTION Soak parts in hot water bath for: 30 minutes at 75°C (167°F) OR CIDEX® OPA Follow manufacturer's instructions for high level disinfection. Immerse for 60 minutes at 20°C (68°F). Rinse thoroughly in potable water.
 SILICONE SEAL	AUTOCCLAVING Gravity displacement-type autoclave. Components should be enclosed in a sterilization peel pouch for 30 minutes; exposure at 121°C (250°F).
 ELBOW	OR STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.
 NON-REBREATHING VALVE	
 HEADGEAR WITH CLIP	Wash in warm soapy water and leave to dry.
 FOAM CUSHION	Replace with new part

THE FLEXIFIT™ 432 FULL FACE MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

Disclaimers: It is the user's responsibility to qualify any deviations from the recommended methods of re-processing. Some discoloration may occur after re-processing.

CONTACT YOUR LOCAL HOME CARE PROVIDER FOR ADDITIONAL INFORMATION.

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