


## Quattro FX components

## Quattro™ FX FULL FACE MASK

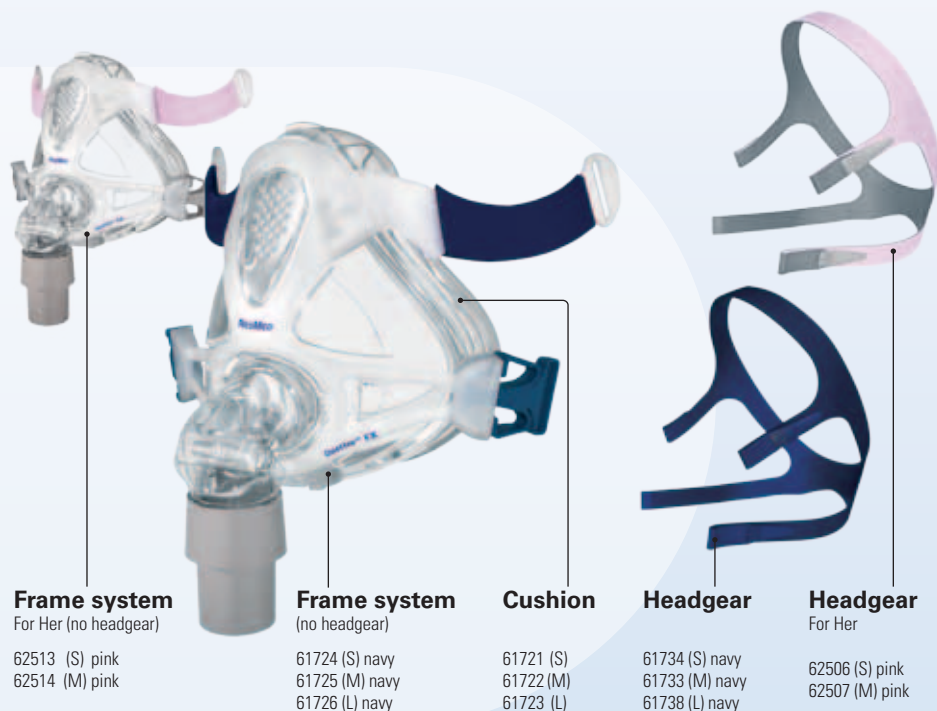

### PRODUCT CODES

Quattro FX Full Face Mask

### Complete System

Size	Part Number
Quattro FX (S)	61700
Quattro FX (M)	61701
Quattro FX (L)	61702
Quattro FX For Her (S)	62501
Quattro FX for Her (M)	62502

*Medium headgear included with all mask sizes*

### Reimbursement Codes

HCPCS	Code Descriptor
A7030	Full face mask used with positive airway pressure device, 1 per 3 months
A7031	Replacement cushion for full face mask, 1 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

### Frame system For Her (no headgear)

62513 (S) pink  
62514 (M) pink

### Frame system (no headgear)

61724 (S) navy  
61725 (M) navy  
61726 (L) navy

### Cushion

61721 (S)  
61722 (M)  
61723 (L)

### Headgear

61734 (S) navy  
61733 (M) navy  
61738 (L) navy

### Headgear For Her

62506 (S) pink  
62507 (M) pink

For details, refer to the **Quattro FX or Quattro FX for Her Full Face Mask User Guide**.

## Fitting



- With both lower headgear clips released, position the cushion at an angle on the nasal bridge.



- Keeping the upper position of the mask in place, swing the lower half of the mask down against the face.



- Hold the mask firmly against the face and pull the headgear over the head.




- Bringing the lower straps below the ears, clip the headgear into the lower part of the spring frame.



- Unfasten the Velcro®, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the Velcro.
- Repeat with lower straps.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

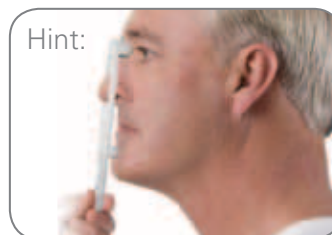

- Start expert fitting by using the available, custom-made Quattro FX sizing tool (61773).

Note: Prior to fitting your Quattro FX mask, remove the orange spring frame support from the headgear and discard.