



B Fitting the Mirage Swift II / Mise en place du masque narinaire Mirage Swift II / Colocación del sistema Mirage Swift II / Colocação da Mirage Swift II



1 ResMed logo faces outward / Logo ResMed tourné vers l'extérieur / El logotipo de ResMed se dirige hacia afuera / O logotipo da ResMed está virado para fora

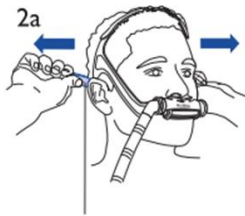

2a Adjust back straps / Ajustez les sangles arrières / Ajuste las correas posteriores / Ajuste as correias traseiras

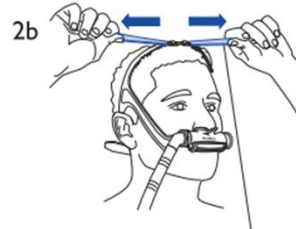


2b Adjust top straps / Ajustez les sangles supérieures / Ajuste las correas superiores / Ajuste as correias superiores



3 Rotate pillow sleeve assembly for comfort and seal / Faites pivoter le corps du masque pour obtenir un ajustement confortable et une étanchéité efficace / Rote el conjunto del manguito de la almohadilla para un ajuste cómodo y sin fugas / Gire o conjunto de manga da almofada para uma melhor vedação e maior conforto



4 Tube retainer (optional) / Bague de fixation du tuyau (facultative) / Retenedor del tubo (opcional) / Retenedor do tubo (opcional)

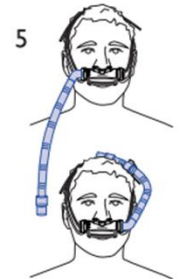

5 Optional tube positions / Posibles del tubo / Posições opt