

## Fitting Guide



### Selecting the correct mask size

- Select smallest fitting template that fits most closely around outside corners of mouth.
- Template should rest in groove between lower lip and chin. Top of template should rest under eyebrow level. This indicates correct mask frame size.
- If line of nose is below dotted template line, use Shallow cushion profile. If above, use Standard cushion profile.

### Fitting the mask

- Move forehead support tab to uppermost position (position 4).
- Disconnect lower headgear clip.



- Place mask on face and pull headgear over head.
- Make sure upper straps pass across forehead and lower straps below ears.
- Reattach headgear clip to mask frame.



- Gently tighten upper and lower headgear straps. The forehead support pads should lightly touch the forehead. Do not overtighten straps.



- Connect one end of air tubing to mask swivel and other end to flow generator.
- Turn flow generator on.
- If air leaks around cushion you may need to change the position of the forehead support tab.



- Loosen upper headgear straps and move forehead support tab downward to next position (position 3).
- Gently tighten upper headgear straps.
- Test all four positions of forehead support tab and adjust headgear straps until you achieve a secure and comfortable fit.