

## Preparing the mask for fitting



Loosen the straps on the headgear by adjusting the Velcro tabs.



Gently squeeze the sides of the StabilitySelector and slide the arm upward to set the forehead support arm to its highest position.



Disconnect one side of the headgear by releasing one ball-and-socket Quick Clip from the faceplate socket.

## Making initial adjustments



Make initial adjustments while sitting.

Place the mask cushion against the face.



Pull the headgear over the head.



Attach the Quick Clip on the disconnected side. For easy attachment of the Quick Clip, use the index finger as a guide to find the position of the socket and press the clip into the socket.