

## Fitting instructions



Evenly adjust the top and bottom headgear straps applying minimal force against the face.



The upper headgear straps should rest above the ears and the lower straps below the ears.



Use the crown strap to adjust the position of the headgear.



With the pressure on, address any leaks by reseating the mask by pulling it away from the face and setting it back into place. Adjusting the upper and lower straps also may help in eliminating leaks.

**Fitting tip:** EasyLife's Auto Seal technology adapts to movement and is designed for a looser fit. Over tightening the mask may hinder its performance.