


Fitting instructions



Disconnect one talon clip.
Connect the flexible tubing from your therapy device to the swivel on the mask. Slide the headgear on over your head. Turn on the airflow before adjustments are made to activate the Auto Seal technology. Reattach the clip.

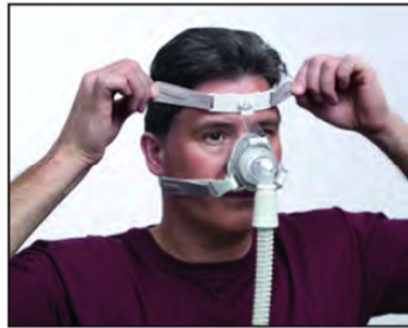

Evenly adjust the side and lower straps. **Do not overtighten:** the cushions should rest lightly on the face.



Adjust the crown strap so the upper straps rest above the ears. The upper and lower straps should be parallel to one another.



Make final adjustments while lying down.

Note: Talon clips do not need to be used. If preferred, they can be removed and the straps can be threaded directly through the mask frame where the clips attach.