

# RESMED

## S9 Escape™

POSITIVE AIRWAY PRESSURE DEVICE



## Welcome Guide


English

**S9**™ Series  
More. Comfort.

**S9**<sup>™</sup> | Series  
More. Comfort.

# Welcome

Thank you for choosing the S9 Escape. Before operating this device, please read the entire Welcome and Information Guides.

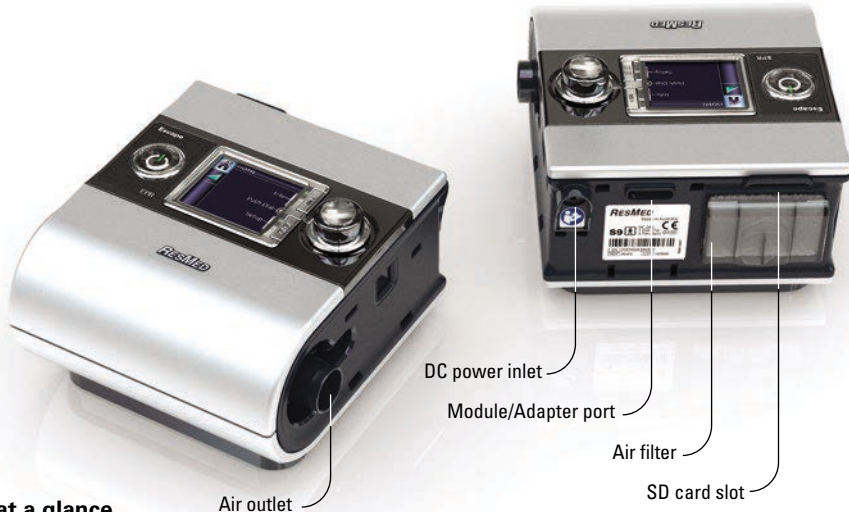

## S9 at a glance

The S9 system comprises the following elements:

- S9 device
- SlimLine air tubing
- 90W or 30W power supply unit
- S9 travel bag
- SD card
- S9 SD card protective folder.


Optional components include:

- H5i heated humidifier
- H5i water tub
- Standard air tubing
- ClimateLine heated air tubing
- ClimateLine<sup>MAX</sup> heated air tubing
- 90W power supply unit
- 30W power supply unit (does not support H5i)
- Power Station II battery pack
- DC/DC Converter 24V/90W.

## Traveling tips

When traveling with your S9 only:

- Ensure you pack the SlimLine or Standard air tubing as the ClimateLine or ClimateLine<sup>MAX</sup> heated air tubing is not designed to connect directly to the S9 device.
- Please ensure you purchase and travel with the approved power cord for the region where you will be using the S9 device.

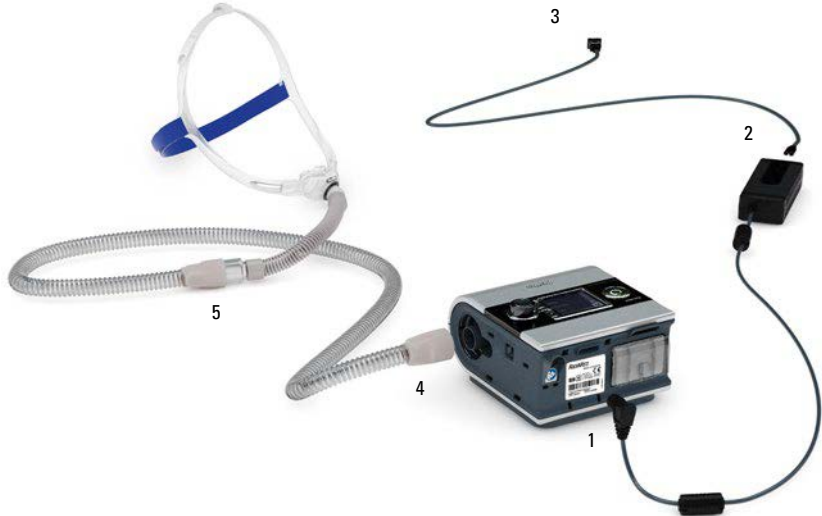

## Setup

1. Connect the DC plug of the power supply unit to the rear of the S9.
2. Connect the power cord to the power supply unit.
3. Plug the other end of the power cord into the power outlet.
4. Connect one end of the air tubing firmly onto the air outlet.
5. Connect the assembled mask system to the free end of the air tubing.

### Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on [www.resmed.com](http://www.resmed.com) on the Products page under Service & Support.

## S9 basics

**Start/Stop button**  
Starts or stops treatment.

**Home menu**

**LCD screen**


**Setup menu\***  
Allows you to make changes to settings.

**Info menu\***  
Allows you to view your sleep statistics.


**Push dial**  
Turning the dial allows you to scroll through the menu and change settings. Pushing the dial allows you to enter into a menu and confirm your choice.


**Key:**


Home 


Humidity level 


Ramp 


Heated tube 

Climate Control 

Start/Stop 

Push dial 

Setup menu 

Info menu 

\*The Info and Setup menus are disabled if S9 Essentials has been enabled by your clinician.





## S9 Essentials


S9 Essentials is designed to make device interaction and menu navigation easier for you. If enabled by the clinician, S9 Essentials disables the Info and Setup functionality so that you can simply start and stop therapy and adjust ramp, humidification and Climate Control.

### Navigating the menus



In general, to navigate the menus:



1. Turn  until the parameter you require is displayed in blue.
2. Press . The selection is highlighted in orange.
3. Turn  until you see the setting that you require.
4. Press  to confirm your choice. The screen returns to blue.



## Getting started

1. Make sure the power is connected.
2. Adjust the ramp time if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time press .

### Notes:

- Once therapy has started a treatment screen is displayed.
- If power is interrupted during treatment, the device automatically restarts therapy when power is restored.

## Cleaning and maintenance

Refer to your mask and humidifier user guides for detailed instructions regarding the care of this device.

### Daily

Remove the air tubing by pulling on the finger grips on the cuff. Hang it in a clean, dry place until next use.

#### Notes:

- Do not hang the air tubing in direct sunlight as it may harden over time and eventually crack.
- Do not wash the air tubing in a washing machine or dishwasher.

### Weekly

1. Remove the air tubing from the S9 device and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang, and allow to dry.
4. Before next use, reconnect the air tubing to the air outlet and mask.

### Monthly

1. Wipe the exterior of the S9 with a damp cloth and mild detergent.
2. Check the air filter for holes and blockage by dirt or dust. Replace the air filter if necessary.

### Replacing the air filter

Replace the air filter every six months (or more often if necessary).

1. Remove the air filter cover from the back of the S9 device.
2. Remove and discard the old air filter.
3. Insert a new ResMed air filter ensuring that it is sitting flat in the air filter cover.
4. Replace the air filter cover.

#### Notes:

- Ensure the air filter and air filter cover are fitted at all times.
- Do not wash the air filter. The air filter is not washable or reusable.

Air filter cover

Air filter







**S9**<sup>™</sup> | Series  
More. Comfort.

368870/1 2012-10

S9 Escape

WELCOME

**EUR 1/3**



Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. **EC REP** ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide.

For patent information, see [www.resmed.com/ip](http://www.resmed.com/ip).

S9, S9 Escape, H5i, SlimLine and ClimateLine are trademarks of ResMed Ltd and S9, H5i, SlimLine and ClimateLine are registered in U.S. Patent and Trademark Office.

© 2012 ResMed Ltd.

